



Uterine Cannula

(latex Free)

Instructions for use

1. Prior to use, identify the position of the uterus (retroverted or antverted)
2. Grasp the anterior lip of the cervix with a vulsellum clamp
3. Measure the length of the intra-uterine cavity with a sound
4. To increase the working length, adjust the acorn down the cannula by twisting and pulling back until it reaches the stop
5. Place the cannula into the cervix in the direction of the uterine cavity
6. If the uterus is retroverted, rotate the cannula 180 degrees
7. Lock the vulsellum clamp into the adjustable lock and pull back to the required tension, fixation is achieved by pulling the lever back
8. If hydrotubation is to be employed, insert the loaded syringe into the syringe port